**12**

**Storage**

16-25°C 60-77°F	≤ 4°C ≤ 39°F	≤ -18°C ≤ 0°F
4h	3d	180d

### NL

#### Algemeen

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### EN

#### General

The mother milk storage bags from Alecto are intended to store, transport and warm mother milk before giving it to the baby.

#### Tips

Write the name, date and volume on the storage bags before filling.

Fill the mother milk bag to a maximum of 220 ml, this is because mother milk expands during freezing.

If you would like to take the mother milk with you on the go, you can keep it in a cool bag with frozen cooling elements for up to 24 hours.

To prevent leakage, make sure that no milk spills on the zip lock closure of the bag.

Shake the bottle gently to mix any separated fat with the milk.

#### Warnings

NEVER warm up the mother milk storage bags in the microwave!

NEVER heat mother milk in a pan of boiling water as vitamins, minerals and other important nutrients will be lost.

NEVER let frozen milk thaw at room temperature! This is because of the formation of bacteria.

The mother milk storage bags are for single use only. The product may **NOT** be reused!

#### Hygiene

Make sure your hands are washed with soap and water before touching the mother milk storage bag. Dry your hands with a disposable tissue.

DO NOT touch the inside of the mother milk storage bag.

#### Material

CPP (Cast Polypropylene). Contains no phthalates, SCCD, BPA or BPS.

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